allard

Support for better life!

Instructions for Use

Mitella Cotton

Art. No. 47101

47101 MITELLA COTTON

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47101 MITELLA COTTON

Carefully read this instruction sheet before use!

ENGLISH

Intended use/Indications: Intended for temporary off-loading for shoulder and arm.

Indications: Post-trauma and post-surgery conditions.

Single patient - multiple use.

Contraindications: None known.

Features:

The shoulder strap can be adjusted by releasing the Velcro strap and tightening or loosening. The two D-rings allow the sling to open wider for use with a cast. Integrated into this sling is a thumb loop to keep arm inside sling.

Adjustments:

This product should only be altered by a medical professional.

Washing Instruction:

Close all Velcro prior to laundering.
Machine wash in 60°C (140°F). Dry flat.
Do not tumble dry or dry clean.

Material Specifications:

Main material: 100% Polyester

Velcro: 100% Nylon

Thumb/Shoulder strap and Edging Tape: 100% Polyester.

D-ring: 100% POM

This product is not flame retardant.

Disposal

Disposal in accordance with local legislation.

Information




Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact:

If any problems occur, remove the support and contact your fitter or medical professional.

Time of use:

Follow your medical professionals guidelines for use and limitations. This product is recommended for single person use only.

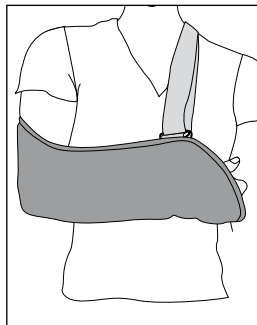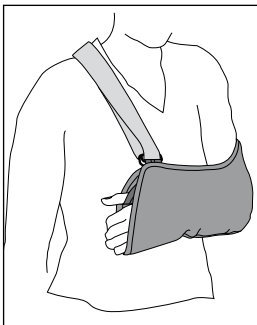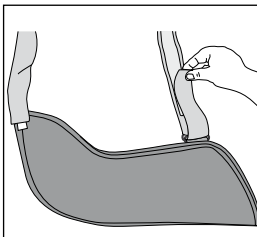Instructions for use

1. Check that the product is complete and without damage.
2. Position the arm into the sling and place the strap over head so the strap comfortably rests on the opposite shoulder.
3. Adjust the strap length so the elbow angle is approx. 90°.
4. Adjust for comfort.

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