VACO® by OPED
paso
FREE

Gebrauchsanweisung
Instructions for use
Mode d'emploi
Istruzioni per l'uso

Manual de instrucciones

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Ihr OPED-Team

VACOpaso Free

Intended purpose: Ready-made therapy shoes to relieve pressure on diabetic foot wounds. You will find readable instructions for use on our OPED homepage.

Safety Advice

- ⚠ The product is single use and should only be used by one patient.
- ⚠ Never apply the product to open wounds and only use under prior medical instruction.
- ⚠ If excessive pain, numbness or sensation changes occur, or if unusual reactions or an unpleasant feeling are experienced while wearing the device, please contact your doctor or orthopaedic technician without delay.
- ⚠ In the case of skin reddening or blistering or should you experience pressure or rubbing, please contact your doctor immediately.
- ⚠ This medical product has been developed for the indications listed and can be damaged by unintended or inappropriate use, by activities such as sport, falls or other accidents, and by liquids that are not compatible with plastics.
- ⚠ Please consult your treating doctor before using it in conjunction with another medical device.
- ⚠ Only replacement parts approved by OPED GmbH may be used.
- ⚠ Please observe the road transport law of your country.
- ⚠ Not made with natural rubber latex. 
- ⚠ Please check the functionality of the VACOpaso Free during regular wound checks. Should you have any questions, please contact your doctor or your OPED retailer.

A How to Use (instruction video can be found at app.oped.de)

- Open the strap fully. Open shoe wide enough to be able to insert your foot. (Fig. 2)
- Insert foot and fasten strap. (Fig. 3)

B Adjusting for right foot

VACOpaso Free can be worn on either foot. The shoe's default setting is for use on the left foot.

- Unthread strap (Fig. 5)
- Turn strap by 90° (Fig. 6)
- Remove strap (Fig. 7)
- Attach the strap to the opposite side as follows: (Fig. 8)
- Insert strap clip into the clasp (Fig. 9)
- Press down strap seal and turn 90° until it snaps. (Fig. 10)
- Pull strap through the first loop (Fig. 11)
- **WARNING:** Do not use loop on the opposite side (Fig. 12)
- Thread strap through the opposite clasp (Fig. 13)
- Close velcro fastening (Fig. 14)

C Customize insole

The patient may only adjust it in consultation with his/her treating doctor.

- The foam material under the insole is divided into seven sections that can be removed individually – this allows the pressure to be relieved on the relevant parts of the foot. (Fig. 16)
- In order to relieve pressure on the regions of the foot affected, remove and dispose of the corresponding sections of foam material from the sole. (Fig. 17)
- Reinsert the insole into the VACOpaso Free.

VACOpaso Free

Information regarding care /cleaning / disinfection / sterilisation

You are receiving this medical product in a hygienically perfect condition. Shoe and VACO12 insole: Clean by wiping down with plastic-friendly cleaning agents; Drying: Air dry, do not dry in a tumble dryer; Disinfection: Wipe disinfection with plastic-compatible disinfectant. Sterilisation: This medical product is not suitable for sterilisation.

Transport conditions / storage conditions / disposal

Storage: Store in a dry and hygienic place and protect from direct sunlight. Please dispose of the medical device in residual or household waste.

Product Characteristics

Wearable on right or left foot. Can be worn as an everyday shoe both indoors and outside.

Table of sizes

	Small	X-Small	Medium	Large	X-Large	XX-Large
EUR	37 – 38	39 – 40	41 – 42	43 – 44	45 – 46	47 – 48
UK	4 – 5	5,5 – 6,5	7 – 8	8,5 – 9,5	10 – 11,5	12 – 13,5
US men		6,5 – 7,5	8 – 9	9,5 – 10,5	11 – 12	12,5 – 13,5
US women	6 – 7	7,5 – 8,5	9 – 10	10,5 – 11,5		
cm	24,5	26	27	28	29,5	31

Indications

- Ulcerations on the foot
- Diabetic foot wounds
- Post-operative care following toe and forefoot amputations

Contraindications

None known

Technical data/parameters

Material composition:

Outer sole (PU), insole board (PA), strap tension (TPE), lining (EVA foam), cover (elastane), velcro (PA); cushion cover (PU layer), filling material (PP spheres), top layer (PE foam)

Warranty

The warranty complies with statutory regulations. The terms and conditions of business of OPED GmbH also apply, where permissible.

Data protection

Information about the processing of your personal data can be found under:

<http://oped.de/support/your-data/>.

Information about the processing of your personal data can be found

<http://oped.ch/kontakt/your-data/>.

In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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OPED UK Ltd
Orthopaedic Appliances Pty Ltd
OPED Medical, Inc.