ENGLISH

CAUTION

1. THE PRODUCT

Optima devices by Molliter are a state of the art product, designed and made with the latest technology, with components that meet the EC regulation 93/42. In spite of this, it is important for the product to be used correctly and not to underestimate the risks that may derive from its incorrect or improper use, including the potential for physical harm and accidents/injuries.

FOR A CORRECT USE

- The product should be sold and used only with a medical prescription or as suggested by a qualified health practitioner.
- Multilayer planters are made of interlocking 3x3 sections (OPTIMA – diab model series, OPTIMA – Clheel model series) or as a single unit (OPTIMA model series – postop, EU.RO.PA, Free). They are meant to be handled only by specialised staff (physicians or specialised orthopaedic technicians).
- Follow the instructions for cleaning and storage.
- Avoid using or pay great attention when using tools, blades or instruments that may perforate, scratch and/or permanently damage the product.
- Pay great attention/be careful when walking on/near wet or slippery surfaces.
- Avoid using the product as a tool or as a toy.
- We advise not to use OPTIMA diab, post-op, EU.RO.PA, Clheel and Free products when driving any motor vehicle.
- Do not put the product next to the fire or sources of excessive heating.
- Check all fastenings to make sure they are tight before using/walking with the product on

2. INDICATED USE

2.1 INDICATIONS FOR THERAPEUTIC USE

OPTIMA DIAB: Diabetic foot ulcer, foot ulcer, rehabilitation after amputation (excepting Chopart-amputation), rehabilitation after operative intervention on tarsus, tarsal fracture, treatment of Charcot foot in chronic phase, surgical intervention on Achilles tendon, ankle fracture, rehabilitative treatment of hemiplegia. The Pneumatic Liner is only to be used in orthopedic indications.

OPTIMA POST OP: Diabetic foot ulcer with vein problems on leg (additional use of kit 3X3), treatment of lateral foot of the injured foot or treatment of foot with risk of ulcer, metatarsal fracture, rehabilitation after toe amputation, stabilization before surgical intervention, toe fractures.

OPTIMA EU.RO.PA: After surgical intervention for hallux valgus, after surgi- cal intervention on hammer toe, after surgical intervention of Morton’s neu- roma, after toe fractures, after nail removal, after wart removal.

OPTIMA FREE: Foot dorsal lesion, after surgical intervention for hallux valgus.

OPTIMA CLHEEL: Diabetic plantar ulcer of the heel, lesion of the tendon, decubitus ulcer. The Pneumatic Liner is only to be used in orthopedic indi- cations.

2.2 LIMITATION

The product presents limitations that derive from its specific use and the characteristics of materials it is made of. If the product undergoes structural variations or negatively affects the patient’s walking, discontinue its use and contact the treating physician.

3. LIMITS OF RESPONSIBILITY

3.1 GENERAL ASPECTS

Our state of the art products are made according to the latest technical and scientific information available in the medical field. Salvatelli s.r.l. shall not be liable or responsible in any way should the product be used under the conditions listed below as “incorrect use”

3.2 INCORRECT USE

“Incorrect use” is intended as being the use of the product in the following conditions:

- failure to abide by the Therapeutic use information;
- failure to follow the manufacturer’s indications and instructions for use of the product;
- failure to abide by or violation of the prevailing safety regulations;
- significant size alteration;
- significant structural alterations with modification/addition of any elements;
- product adaptations that significantly deviate from applicable regulations/ indications;
- using the product while playing sports, even for a short time;
- cleaning with unsuitable products;
- incorrect maintenance;
- product abuse;
- driving motor vehicles while using OPTIMA products.

4. USER INSTRUCTION

4.1 GENERAL ASPECTS

The product is engineered to maintain its original characteristics through its life-cycle (a season on average) when it is used and stored as indicated; its actual life span may vary depending upon the actual environmental conditions and use. The product may be used as is, or with the optional plantar kit as explicitly indicated above. For the product to perform correctly, it is important not to modify its structure and support points.

4.2 PRODUCT CLEANING AND STORAGE

To clean the product use lukewarm or cold water, clean with neutral soap and rinse thereafter avoiding using any fabric-aggressive liquids such as alcohol, laundry detergents, bleach, ammonia, or solvents. Avoid the use of abrasive tools such as brushes, steel wool, sand paper or knife blades to eliminate spots. Salvatelli srl is NOT responsible for any damage caused by abrasives, aggres- sive products or by direct exposure to heat sources.

5. PRODUCT DISPOSAL

Only some of the components of OPTIMA by Molliter devices are eco-friendly. For this reason, it cannot be treated as regular trash; due caution should be exercised when disposing of it. We recommend that you should check the local and National regulations for the proper disposal of the product.

6. PRODUCT IDENTIFICATION DATA

| DATA | DETAILS |
|-------------------------------|---|
| Year of production | 2014 |
| Dimensions | according to the article size and model |
| Weight | according to the article size and model |
| Classification of the product | class I Norm 93/42 EC |
| Production mark | see code in front of each box |
| Size | S (34-35-36) M (37-38-39) L (40-41-42) XL (43-44-45) XXL (46-47-48) |

Optima devices and plantar kits by Molliter are entirely made in Italy

7. GUARANTEE

Our guarantee is limited to cover the materials and manufacturing defects of the product in accordance with the Law. This includes the replacement of products found to be defective by the manufacturer.

The guarantee is not applied to the following cases:

- damage caused by transport and not contested by the purchaser at the moment of delivery;
- repairs executed by entities other than the manufacturer;
- normal wear and tear of the product;
- damage caused by the purchaser, as a consequence of incorrect use, main- tenance or storage of the product.

7.1 RETURN PROCEDURES

Defective articles may be returned by the distributor or the customer after Salvatelli s.r.l. authorises it. In this case, the items should be placed back in their original box with proof of purchase and only with prepaid postage. Should the Company not deem it was responsible for the defect, the product shall be sent back at the expenses of the distributor or client.

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