**46025 VISSION
NEO THUMB
SHORT**

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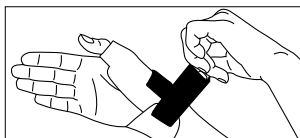
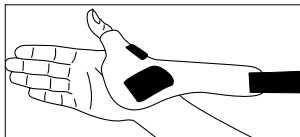
Indications / Intended Use

The products are intended to support the CMC and MCP joints. CMC I arthritis, rheumatoid arthritis, after minor trauma, hyper mobility in thumb and wrist, after arthropod plastic surgery or in conditions such as fatigue. This product is only for one person use.

Contraindications

Fractures.

Adjustments

This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

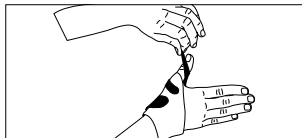
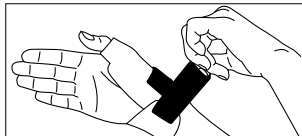
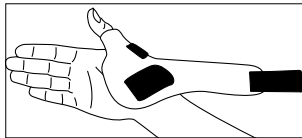
This orthosis is made from Nylon and SBR (Latex Free Neoprene). This product is not flame retardant.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use

1. Check that the support is complete and without damage.
2. Attach the thumb strap and place the opening over the thumb with the orthosis towards the back of the hand.
3. Pull the strap around the wrist and secure.
4. Adjust the strap around the thumb so it feels comfortable.
5. Adjust for comfort.

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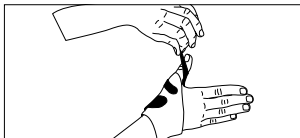
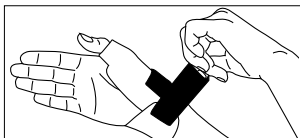
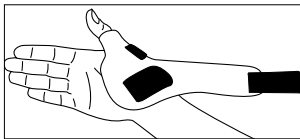
Support For Better Life • www.allardsupport.com

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Camp Scandinavia AB, Karbingatan 38