# ASPEN<sup>®</sup>CTO

CERVICAL THORACIC ORTHOSIS  
ZERVIKAL-THORAKALE ORTHESE  
ÓRTESIS TORÁCICA CERVICAL  
ORTESI CERVICO-TORACICA  
ORTHÈSE CERVICO-THORACIQUE  
CERVICALE THORACALE ORTHÈSE

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







SUPINE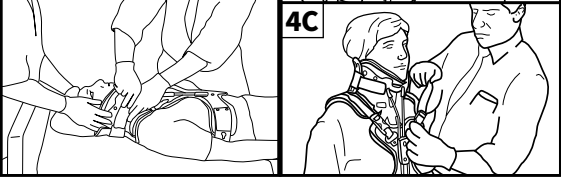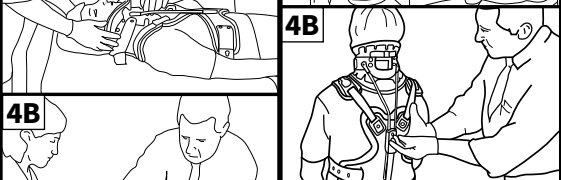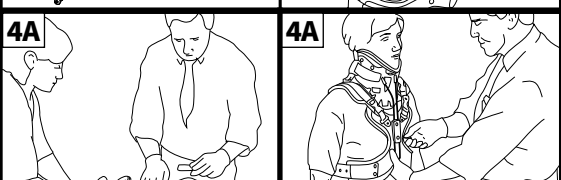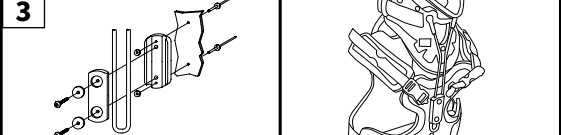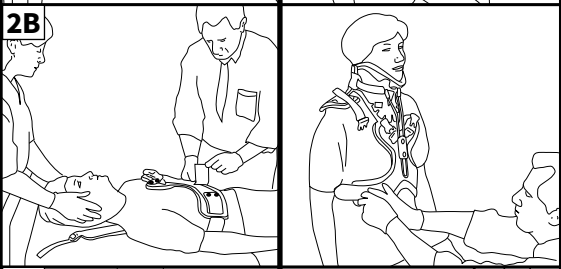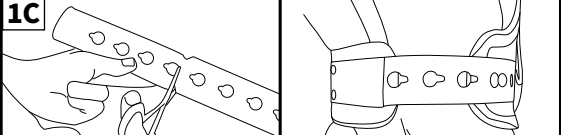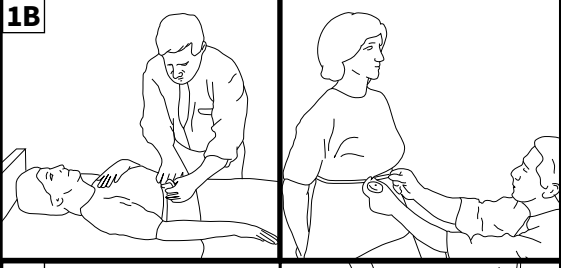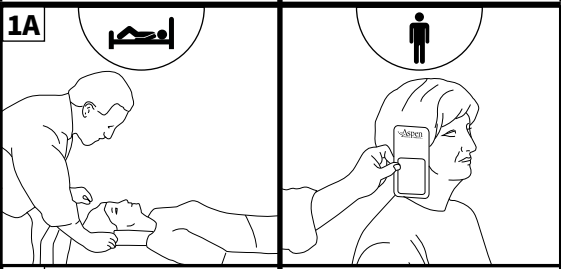

ENGLISH

**1. SIZING**  
A. Use the Aspen sizing guide to measure the patient for the correct size Aspen Collar. The sizing guide can be found at aspenmp.com.  
B. Measure the chest circumference 1 inch below the tip of the Xiphoid process.  
C. For circumferences smaller than 39", trim the ends of the belt panels at the notch.  
D. Attach the belt by aligning the keyhole over the aluminum rivet and pull to lock in place. Do this on both sides.

**2. FITTING**  
A. Fit the collar to the patient as described in the Aspen collar instruction sheet. The instruction sheet can be found at aspenmp.com.  
B. (Supine): Slide the CTO under the patient while compressing the mattress cover. Layroll the patient if needed.) Line up the control clamp to the centerline on the chest and attach the hook and loop.  
C. Snap the shoulder straps into place and loosely tighten. Bars may be bent to accommodate patient anatomy.  
NOTE: To relieve pressure at the front on the collar, bend bars away from the patient.

**3. CONTROL CLAMP**  
Use a phillips head screw driver to loosen screws to adjust the height of the bars. Tighten the clamp after adjustment.

**4. ADJUSTMENT**  
A. Slide the support posts on the front of the CTO upward until the patient's head is properly supported. Tighten the screws on the support clamp.  
B. Adjust the collar back in the same manner. (Hint: the higher the back of the collar, the more effective the motion restriction.)  
C. Check for symmetry and tighten shoulder straps removing slack.

**NOTES**  
• The Aspen<sup>®</sup> CTO can be altered to mirror changes in the patient's condition. For supine patients where posterior support posts are desirable, a pillow may be used under one shoulder to minimize pressure caused by the posterior support post.  
• Be aware that excessive tightening of the shoulder straps will cause traction.  
• Do not use a full size screwdriver to tighten or adjust the support post clamps while the device is on the patient.  
• When advancing to a 4-post configuration, ensure that the post strap covers the support post.  
• For kyphotic patients or for others with postural abnormalities, it is possible to bend the support posts for a better fit. To accomplish this, use a pair of bending irons or remove the posts from the clamp and bend using a vise to ensure that they remain symmetrical. Be careful not to weaken the posts by bending them repeatedly or by bending them more than 45 degrees.  
• Replacement pads are available from your doctor, local orthotic and prosthetic facility, or by calling Aspen Medical Products at 800-295-2776.

**INTENDED USE**  
To provide motion restriction of the cervical spine.

**INDICATIONS**  
Decompressive procedures, Fracture management, Post-surgical stabilization, Whiplash, Spains/strains, Cervicogenic headache, Cervical disc syndrome, Musculoskeletal pain, Radiculopathy.

**CONTRAINDICATIONS**  
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

**PRECAUTIONS**  
READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

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