**SELECTION
BACK WIDE
35402**

SELECTION[®]

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Intended Use

Selection Back Wide is intended to be used for moderate Back Pain and weakness. This model is especially suitable for persons with big belly. The front is in two parts, which facilitates donning and allows for varied height of the front piece.

Indications

Back Pain, Back insufficiency. Single Patient-multiple use.

Contraindications

Sciatica with leg or foot numbness or weakness.

Adjustment

This product should only be altered by a medical professional person.

Warning

Do not pull the straps too tight, as this can cause poor circulation and/or skin irritation. If you feel any irritations or other problems, take it off immediately. Check skin on a daily basis.

How to use the support

Check that you have the correct size. Some conditions can result in swelling that can mean variations in size. If this is the first time you are using a support it is important to start slowly. Use the support for a short period of time the first day, then increase the use gradually. The support should not be used overnight.

Material specifications

72% Polyester, 28% Spandex. Stays, Velcro and buckles made of nylon. This product does not contain Latex. This product is not flame retardant.

Wash instruction

Close all Velcro prior to laundering. Machine washable in 40°C (104°F). Dry flat. Do not tumble dry or dry clean. Use a washing bag.

Disposal

Disposal in accordance with local legislation.

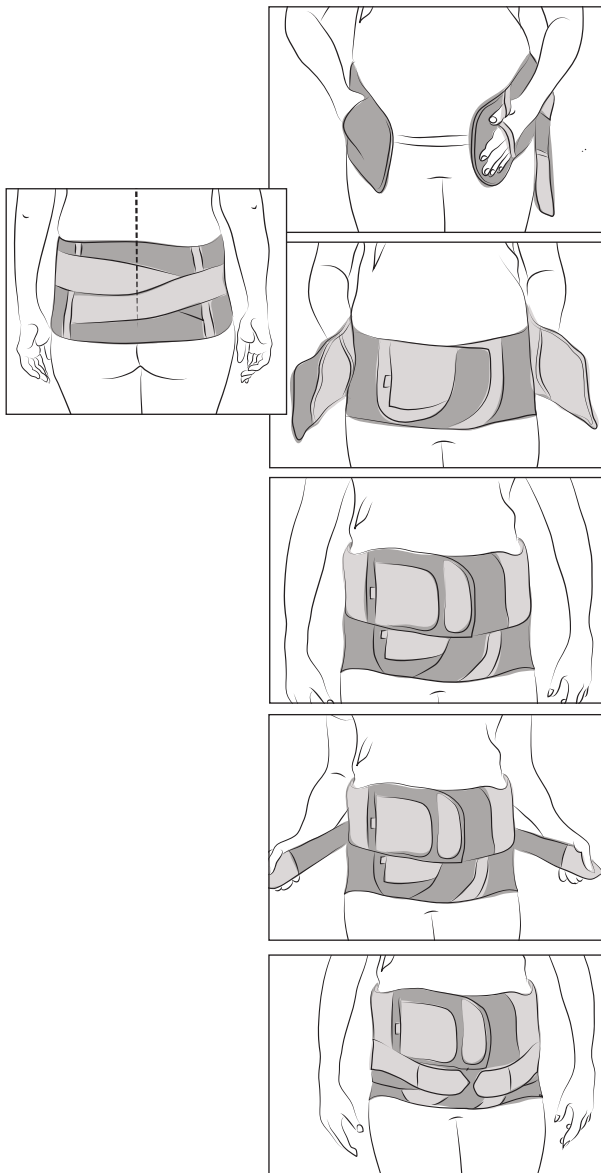
Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to contact

If any problems occur please contact your orthotist or fitter.

Instructions for use

1. Check that the support is complete and without damage.

2. Loosen the two stabilizing straps so that the cross in the back becomes loose. Also open the front closure and separate all parts.

3. Put the support around your body and secure that it is centered over the spine. Begin with closing the lower section on the front using the pocket and the strap. Then close the upper section of the front on top of the lower section.

4. Grasp hold of the two stabilizing straps, pull them forward and secure on the front.

5. To remove the support: First open the two stabilizing straps.

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Camp Scandinavia AB, Karbingatan 38