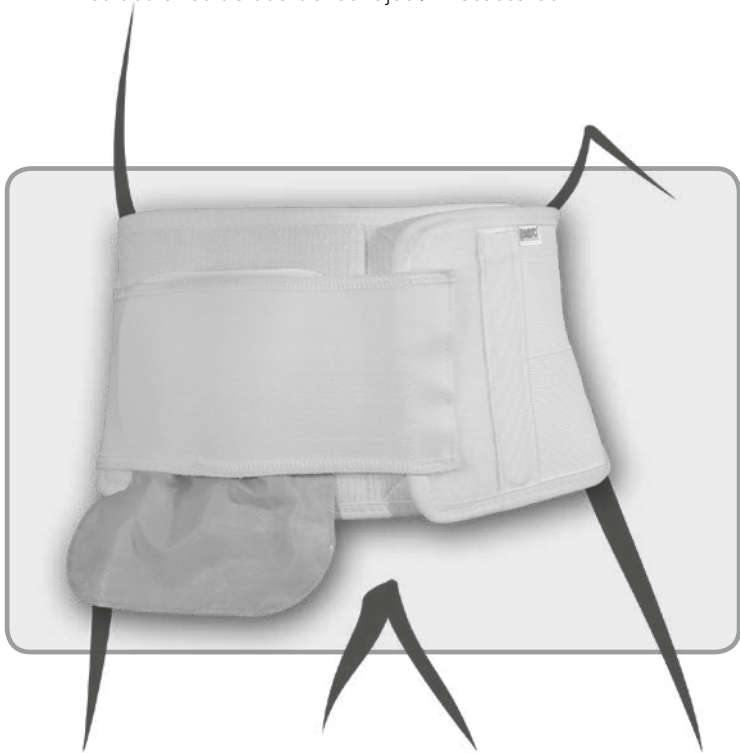
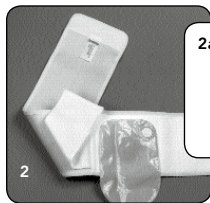
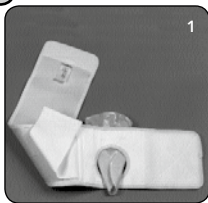
Basko
Healthcare

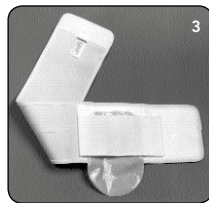
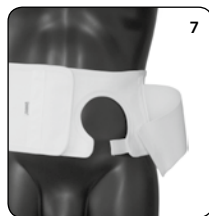
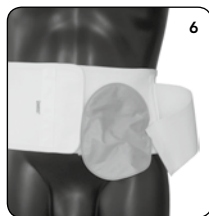
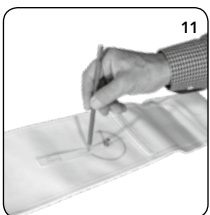
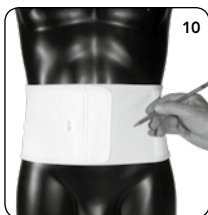
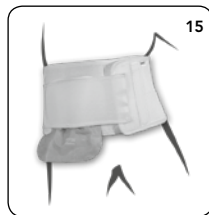
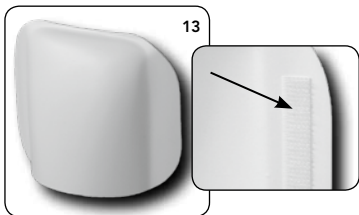
Stomacare

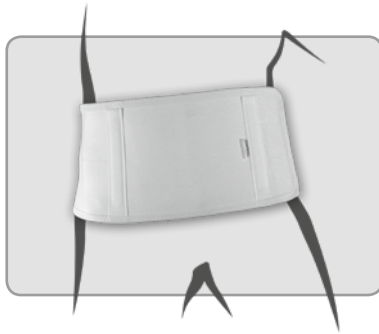
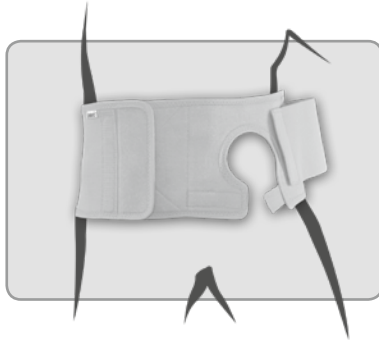
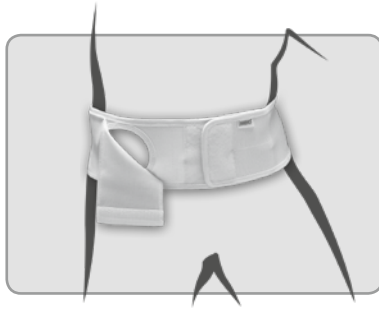
Gebrauchsanleitung Bandagen / Protector
Gebruiksaanwijzing bandagen / Protector
User instructions Support Belts / Protector
Användarinstruktioner Stomibandage / Protector
Bruksanvisning Stomibrokkbelte / Protector
Brugerinstruktioner Stomibælte / Protector
Sovitusohjeet / Avanneliivi / Protector
Istruzioni d'uso della fascia / Protector
Mode d'emploi bandages / Protector
Instrucciones de uso de las fajas / Protectoras

A

2a

**B****C****D****E**

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A Standard Models**Fitting the belt**

- 1) **Fig. 1:** Pull to open the Velcro prolapse cover. Fold the stoma pouch into a point followed by pulling the pouch through the stoma opening from the inside of the stoma bandage.
- 2) **Fig. 2:** The stoma pouch is now completely pulled through the opening, with the wafer remaining at the inside of the belt.
- 3) **Fig. 2a:** Optionally the Protector (accessory) can be mounted now. For fitting, see below.
- 4) **Fig. 3:** Pull on the bandage at the desired tension and close the Velcro fastenings. Close the prolapse cover over the stoma bag.

B EasyOpener Models**Fitting the belt**

Follow the steps 1 to 4 as described in section A (Standard Models).

Stoma bag replacement

- 1) **Fig. 4:** Loosen the Velcro prolaps cover.
- 2) **Fig. 5:** If present, remove the Stomacare Protector from the belt.
- 3) **Fig. 6:** Slightly lift up the stoma pouch and open the special closure underneath the stoma opening.
- 4) **Fig. 7:** Gently pull the sticking edging of the stoma pouch from the skin and remove it. For re-fitting a stoma pouch follow the steps as described previously.

C Irrigation Models**Fitting the bandage**

- 1) **Fig. 8:** Pull the belt at the desired tension.
- 2) **Fig. 9:** Close the Velcro closure.

D EasyCut Models**Applying stoma aperture**

- 1) Open the frontal Velcro closure and position the belt at the desired tension onto the patient's body. Make sure the Velour panel is spaciouly positioned around the stoma.
- 2) **Fig. 10:** Mark the center of the patient's stoma with a marker onto the Velour panel. Make sure sideways sufficient Velour is left for securing the Prolapse cover and / or Protector.
- 3) **Fig. 11:** Remove the belt and lay them down flat. Now use the supplied template and mark, depending the pouch system, a circle or oval on the Velcro panel.
- 4) **Fig. 12:** Use sharp scissors to cut the marked hole in the belt. The velour material does not fray and is sufficiently rigid for optimal support.

Fitting the belt

Fold the stoma pouch into a point and pull it through the stoma opening from the inside of the belt. Now, follow steps 2 to 4 as described above in section A (Standard Models).

E Protector

The Stomacare Protector is an accessory and offers additional protection during activities (e.g. sport, car driving) against pressure and the effects of external forces on the stoma. The Protector can be fitted to all Stomacare models.

Fig. 13: On the inside of the Protector there are Velcro fastenings on left and right with which it can be attached to the front of the Stomacare belt.

Fig. 14: osition the Protector around the stoma opening of the belt. Make sure the stoma bag is fully enclosed at each side. The stoma pouch must protrude from the underside of the Protector.

Fig. 15: Pull the prolapse cover to the desired tension over the Protector and fasten the Velcro strip. The Protector must be removed before washing the bandage.

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ENG Stoma pouches: The stoma openings are designed to fit the most common single- and two-part stoma systems. **Washing instructions:** The Stomacare belts from Basko Healthcare are long-lasting and high quality products. We advise you to wash the belt in lukewarm water with a fine detergent. To avoid damage to the material, the Velcro fasteners must be closed before washing.

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